


**Stealth Products**

## Armrest Electronic Mounts



### USER GUIDE

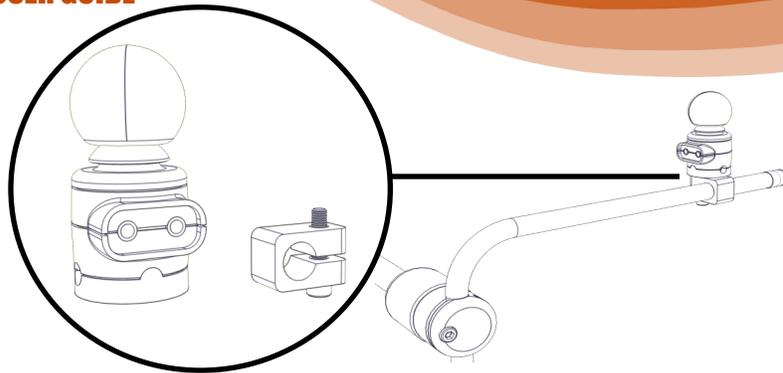

## GAT461M

### Intended Use

The GAT461M mounting hardware allows a Multi Joystick to be mounted directly to the arm of any Gatlin 400-, 405-, or 406-Series mid-line positioning mount. The GAT461M is designed to offer multiple points of adjustability, including joystick angle and horizontal alignment.

Our products are designed, manufactured and produced to the highest of standards. If any defect in material or workmanship is found, Stealth Products, LLC will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty.

No person is authorized to alter, extend or waive the warranties of Stealth Products, LLC.




Stealth Products, LLC  
104 John Kelly Drive, Burnet, TX 78611  
Phone: (512) 715-9995      Toll Free: (800) 965-9229  
Fax: (512) 715-9954      Toll Free: (800) 806-1225




MDSS GmbH  
Schiffgraben 41




## Installation Instructions

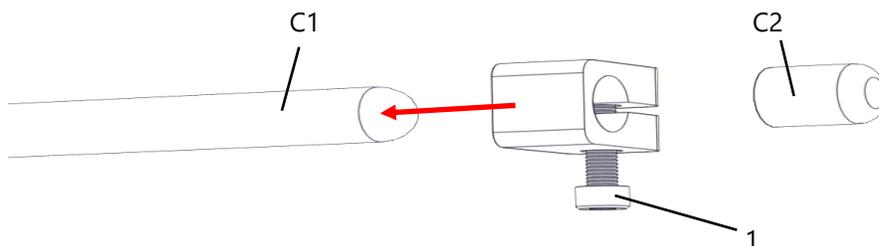

**Step One:** Remove bar plug (C2) from Gatlin arm (C1). Loosen clamping screw (1). Slide clamp (A) onto the arm (C1).

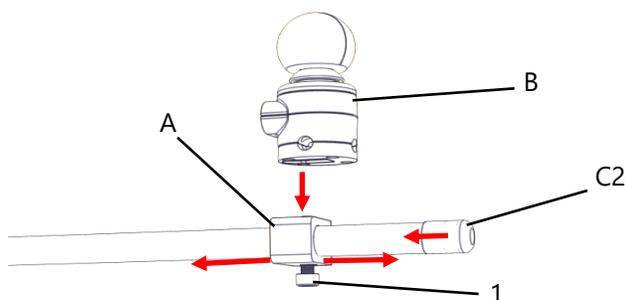

**Step Two:** Slide clamp (A) to desired horizontal position. Replace bar plug (C2). Hold clamping screw (1) and position Multi Joystick (B) in place over the clamp (A).

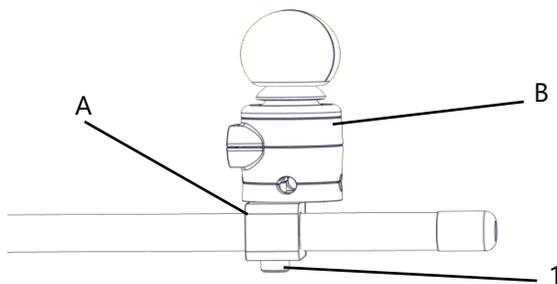

**Step Three:** Tighten the screw (1) with a 4mm hex key, taking care not to over-tighten. Mount clamp (A) and tighten to Multi Joystick (B).

### **WARNING**

This product is designed to be fitted, applied and installed exclusively by a healthcare professional trained for these purposes. The fitting, application and/or installation by a non-qualified individual could result in serious injury.