




Push and Transfer Gloves



Push and Transfer Gloves have an open finger, half thumb, wrap-around design that makes the gloves easy to put on and take off. Made of leather with padded palms and wrists, the gloves help prevent wear and tear on user's hands throughout a long day of active chair use. Three inches of hook-and-loop closure provide a highly adjustable and secure fit for most hand sizes.

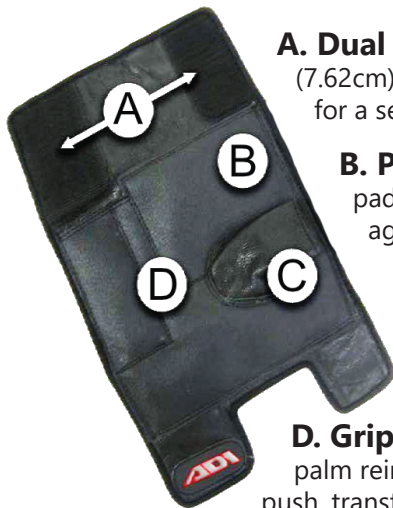

A. Dual Hook-and-Loop: 3 in. (7.62cm) of dual hook-and-loop closure for a secure fit.

B. Padded Palm: Reinforced padding offers extra protection against pressure and abrasion.

C. Open Finger: Half-thumb design enables independent utilization by persons with limited hand function.

D. Grip Options: Leather or rubber palm reinforcement to enhance grip for push, transfer, and braking.

To use the Push and Transfer Gloves:

1. Open the glove by separating the hook-and-loop closure to expose the interior of the glove. Insert your thumb into the half-thumb section of the glove.
2. Pull the "loop" flap of the glove around to the back of your hand. Bring the "hook" flap across your wrist and press to secure it. Adjust the fit for comfort.
3. Pull the other "hook" flap across the back of your hand and press it to the "loop" flap. Adjust the fit for comfort.
4. Reverse the procedure to remove the glove.

Notice: One size fits most. Before each use, it is important that you check the **Push and Transfer Gloves** for any damage or wear.

For product information visit: <https://stlpro.site/stealth-docs> and enter "Push and Transfer Gloves" in the search bar.