


## Stealth Positioning™ Cam Lock



*The Stealth Positioning Cam Lock is intended to provide a mechanically stable and secure connection point for support devices using 1", 1.5", or 2" webbing straps.*

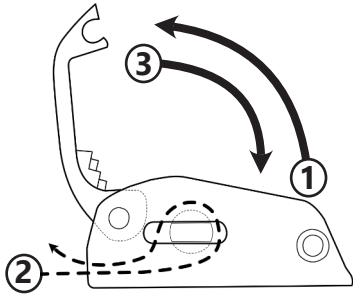

1. Lift latch.
2. Thread strap.
3. Close latch.

### Installing the Strap

If the latch is closed, use a screwdriver or other similar tool to release it. With the latch open, slide the strap under the latch, around the serrated rod inside, and back under the latch. Properly adjust the tension of the strap, and press the latch to the closed position, where it will lock into place when fully closed. It will hold the strap securely in position and will require a screwdriver or other similar tool to open again.

### Adjusting the Strap

Using a screwdriver, or other similar tool, pry the latch until it snaps open. The strap should loosen, and can be adjusted. When satisfied with the adjustment, close the latch until it clicks, securing the strap in the new position.

### Installing the Cam Lock

The Cam Lock is mounted directly to a chair rail or tube. Using a flat head screwdriver, or other suitable prying device, open the latch. Use a button head screw of sufficient size to fasten the Cam Lock to the mount point on the chair, ensuring the latch is clear to close fully once installed.

